



GTIN: 7640178011052

#SMT1020-EL

White Elegant

Tooth Whitening Gel/ Dental Cosmetics

WHITE ELEGANT is two-Step tooth whitening gel to remove unsightly tooth discolouration.

Features and Benefits:

- Two-Step tooth whitening gel
- Easy to use
- Cheek retractor provides you with comfort
- It has Peppermint oil flavour
- No need for light activator
- 2 x 4 ml pens

Ingredients:

- Pen 1. Carbamide Peroxide 6%, Peppermint oil, Carbomer, Exp.
- Pen 2- Carbomer, Copper Salt, Exp.

Instruction For Use:

- 1-Brush your teeth
- 2-Place the cheek retractor in your mouth
- 3-Use Step 1 Pen, press two drops onto each tooth in circular motion
- 4-Use Step 2 Pen, press two drops onto each tooth and mix in circular motion
- 5- Allow 5-10 minutes for bleaching
- 6-Wipe your teeth with tissue
- 7-Remove the Cheek retractor
- 8-Rinse your mouth under running tap water.
- 9- Brush your teeth
- 10-Smile

Component:

White Elegant

Art.No. SMT 1020-EL

Pen 1- Whitening Gel (4ml)

Pen 2- Activator Gel (4ml)

Cheek retractor 1x



Oxidising agent



SwissMediTec GmbH

Bahnhofstrasse 42

6162 Entlebuch

Switzerland

Tel.: +41 41 481 00 86

+41 79 482 32 98

www.swiss-meditec.ch

Email: info@swiss-meditec.ch