



GTIN: 7640178010789

Medi White Home 16%-Kit
TOOTH WHITENING GEL BASED ON CARBAMIDE
PEROXIDE



SwissMediTec GmbH

Businesspark
Bahnhofstrasse 42
6162 Entlebuch
Switzerland
Tel.: +41 41 481 00 86
+41 79 482 32 98
www.swiss-meditec.ch
Email: info@swiss-meditec.ch

Medi-white Home 16% is dual barrel syringe formulated for aesthetic whitening discolored teeth prior to placement of composite, veneers, and/or crowns. It is effective in removing some or all internal tooth discolorations due to congenital, systemic, pharmacologic, traumatic, etc., factors as well as aging. It is even successful at resolving staining from fluorosis and tetracycline.

Composition:

1-(EDTA), Peppermint oil, Carbopol and Carbamide Peroxide in a water soluble glycol base.
2-Carbopol, Copper salt, water

Properties:

-Removes surface stains and plaque from teeth.
-Enhances whitening treatments by optimizing the pH in the mouth. Mildly abrasive to enamel / dentin High cleaning and polishing values.

Not intended for regular use.

Procedure:

Preparing the syringe and accessories

- 1-Insert cheek retractor into mouth
- 2-Attach a mixing tip or to the syringe in the usual manner.
- 3-Extrude small quantity of **Medi White Home II** gel onto each tooth.
- 4-Use micro brush in a circular motion to introduce the gel within teeth.
- 5-Bleach for 5-10 min.
- 6- Whip your teeth with tissue
- 7-Remove the **cheek retractor**
- 8-Rinse your mouth with water
- 9-Brush your teeth gently
- 10-Clean the Cheek retractor
- 11-Smile

Precautions:

In case of uncomfortable irritation or significant sensitivity, stop the treatment and use lower concentration or shorter treatment

Medi-White Home contains carbamide peroxide and is corrosive, causes burn. Wear appropriate protective clothing, gloves and eye protection. In case of accident or if you feel unwell, seek medical advice.

Storage:

Keep refrigerated (2°C to 8°C)

Made in Switzerland



Oxidising agent Refer to instruction manual