



GTIN: 7640178010888

Medi Polish

Art.No.3047

Teeth Polishing Paste and stain eraser

Introduction:

Medi Polish is a diamond impregnated polishing paste used both intra and extra orally for achieving a high luster polish on both the restoration and natural tooth surface.

Features & Benefits

*Ultra fine diamond particles (Zirconium Oxide) in a smooth paste.

*20% super fine diamond particles

Provides high luster polishing for any restoration

*Smooth, Peppermint oil flavored intraoral paste

*Optimal viscosity and excellent handling

Indications:

1. High luster polishing of restorative materials (porcelain, direct composite, lab composite, metals etc.)

2. Polishing and cleaning of natural teeth

Procedure:

1-Put some **Medi Polish** paste inside the cup

2-Put some **Medi Polish** directly on teeth

3-Lean the cup on teeth and start the dental polisher. Start the polisher only when it is inside the mouth to avoid splashing around.

4-Do circular slow movements, use a moderate pressure for two to three seconds for every tooth. When the pressure is too excessive, the polisher will stop for enamel safety.

5-After the operation, wash both teeth and the polisher cup.

Composition:

Sorbitol, Calcium Carbonate, Aqua, Propylene Glycol,

Sodium Saccharin, Zirconium Oxide, Peppermint oil.

For dental professional use only

Made in Switzerland

Manufactures address



SwissMediTec GmbH

Businesspark

Bahnhofstrasse 41

6162 Entlebuch

Switzerland

Tel.:+41 41 481 00 86

+41 79 482 32 98

www.swiss-meditec.ch

Email:info@swiss-meditec.ch